- Schutzkappe | protective cap | capuchon de protection | copercchio di protezione | tampa protectora | tampa protetora | beschermkap | pokrywa ochronna | fodal
- Heizplättchen (Anwendungsteil) | heating plate (applied part) | plaque chauffante (partie appliquée) | piastra termica (parte applicata) | placa calefactora (superficie de para tratamiento) | placa de aquecimento (área de aplicação) | verwarmingsplaat (toepassingsdeel) | płytka grzewcza (obszar zastosowania) | värmeplatta (användningsyta)
- USB-C / Lightning Stecker | plug | connector | connecteur | encheufe | plugue | conector | złącze | kontakt

**Download App**  

**Anwendung | application | aplicación**  
 applicazione | uso | aplicação | toepassing | zastosowanie | användning

**Zeichenerklärung | signs & symbols | signes & symboles | segni & simboli | explicación de los símbolos | explicação dos símbolos | explicación dos símbolos | verkling van de symbolen | wyjaśnienie symboli | teckenförklaring**

- Transport & Storage**  
 Zulässige Transport- und Lagertemperatur | Permissible storage and transport temperature  
 I Température de transport et de stockage autorisée | Temperatura di trasporto e stoccaggio consentita | Temperatura permitida para almacenamiento y transporte | Toegestane transport- en opslagtemperatuur | Dopuszczalna temperatura transportu i przechowywania | Tillåten transport- och förvaringstemperatur
- Operating**  
 Zulässige Betriebstemperatur | Permissible operating temperature | Temperature de fonctionnement admissible | Temperatura di funzionamento consentita | Temperatura permitida para el funcionamiento | Temperatura de operação permitida | Toelaatbare bedrijfstemperatuur | Dopuszczalna temperatura pracy | Tillåten användningstemperatur

- Zulässige Luftfeuchtigkeit und zulässiger Luftdruck für Betrieb, Transport und Lagerung | Permissible humidity and air pressure for operation, transport and storage | Humidité et pression de l'air admissibles pour l'utilisation, le transport et le stockage. | Umidade e pressão de ar consentida per fuso, il trasporto e lo stoccaggio. | Humedad y presión permitida para uso, transporte y almacenamiento | Umidade e pressão de ar permitidas para uso, transporte e armazenamento | Toelaatbare vochtigheid en luchtdruk bij gebruik, vervoer en opslag | Dopuszczalna wilgotność i dopuszczalne ciśnienie powietrza podczas użytkowania, transportu i przechowywania | Tillåten luftfuktighet och lufttryck för användning, transport och förvaring

- Gebrauchsanweisung befolgen! | Follow the instructions for use! | Suivez les instructions d'utilisation! | Seguir le istruzioni per l'uso! | Sigla las instrucciones de uso! | Sigas as instruções de uso! | Volg de bruiksaanwijzing! | Postępuj zgodnie z instrukcją obsługi! | Folj bruksanvisningen!

- max. 5.25 V, max. 500 mA (heat it USB-C)  
 max. 4.2 V, max. 300 mA (heat it Lightning)

- Maximale Anwendungsdauer und -temperatur | Maximum treatment duration and temperature | Durée et température maximales du traitement | Durata e temperatura massima del trattamento | Máxima duración de tratamiento y temperatura | Tempo e temperatura máximos de aplicação | Maximalle toepassingstijd en -temperatuur | Maksymalny czas trwania i temperatura zabiegu | Maximal behandlingstid och -temperatur

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The intended purpose of **heat it** is to locally heat an affected skin region to temperatures of about 51 °C for a few seconds („hyperthermia“). The device is used in combination with a smartphone and controlled via the **heat it** app.

- 1. Intended Use & Operating Principle**  
**heat it** is a medical device to treat itching and pain caused by insect bites and stings, for example caused by:
- mosquitoes
  - horseflies
  - wasps
  - bees

The application can relieve itching and pain quickly and for a long time. Treatment is possible both immediately after the sting and if symptoms occur later.

- 2. Installation**  
**heat it** only works in conjunction with a suitable smartphone with CE marking.
- Scan the QR code or search for the **heat it** app in your app store.
  - Install the **heat it** app (Version 2.5.0 or newer)
  - Plug **heat it** into the charging port of your smartphone.
  - The app signals that **heat it** can be used.

- heat it** is available in different variants, please note:
- iPhone 6s to 14 require the **heat it Lightning** variant.
  - Android smartphones (5.0 or newer) and iPhone (15 or newer) require the **heat it USB-C** variant.
  - The smartphone must support the OTG standard. For some smartphones, OTG must be activated via the settings menu (e.g. on Oppo, OnePlus, Vivo devices).

- 3. Treatment selection**

**Treatment duration:**  
 Choose short, medium or long (4 / 7 / 9 seconds)

**Treatment temperature:**  
 Choose the **Child** setting (48,5°C) for children and adults with sensitive skin, and the **Adult** setting (51,5°C) for non-sensitive adults. The treatment temperature can be reduced by 1,5°C for **sensitive** skin areas.

In order to get to know the treatment principle, the settings **short, child** and **sensitive** must be selected for the first application. For subsequent treatments, the settings should be selected based on individual sensitivity.

- 4. Application**  
 The app guides you through the simple application of **heat it**:
- Plug **heat it** into the smartphone. The **heat it** app opens and the LED on **heat it** pulses green.
  - Select the appropriate settings in the menu, please see **section 3**.
  - Press the start button or a volume button. **heat it** will now preheat for a few seconds. **Do not press the heat it on your skin during this time.**
  - The completion of the preheating process is indicated on the screen. Holding the smartphone, press the **heat it** heating plate onto the bite. To stop the treatment prematurely, lift **heat it** off the skin.
  - The end of the treatment is indicated on the screen. Lift the heating plate off the bite.
  - Although this will usually not be necessary, the treatment can be repeated after a pause of **at least 2 minutes**.
  - To switch off **heat it**, disconnect it from the smartphone.

- 5. Other Information**
- Remove bee stinger before treatment with **heat it**.
  - **heat it** may be used with the assistance of an adult on children aged 3 and older. It can be used independently by children aged 12 and above
  - In case of any uncertainty and for the first application, the symbols **Short, Child** and **Sensitive** must be selected. The same applies for the treatment of body parts that have not yet been tested.
  - Special maintenance of **heat it** is not necessary, but the heating plate should be cleaned regularly with a disinfecting cloth. Disconnect **heat it** from the smartphone before cleaning.
  - Do not clean **heat it** under running water or immerse it in water.
  - A red pulsating LED indicates that **heat it** cannot connect correctly to the app. Please ensure that a current version of the app is installed and that the **charging socket is free of dirt**.
  - **heat it** is a medical electrical equipment which becomes a medical electrical system in combination with the smartphone.
  - Please make sure that **heat it** is firmly seated in the protective cap.
  - Storage instructions: When not in use for extended periods of time, store **heat it** away from direct sunlight and in a dry place. Disconnect **heat it** from the smartphone and firmly seat it in the protective cap immediately after use and store it separately.

- 6. Contraindications**
- Do not use **heat it** on damaged skin (e.g. sunburns) and on wounds.
  - Do not use **heat it** on damp skin, or on skin that has previously been treated with ointments, creams or gels.
  - **heat it** is designed exclusively for use on the outer skin. Do not use **heat it** in body orifices, on mucous membranes, gums, nail beds, eyes or external auditory canals.
  - **heat it** must not be used on particularly sensitive skin regions such as poor circulation (poor circulation of the limbs due to diabetes, varicosis, atherosclerosis, scleroderma, Raynaud's syndrome, etc.).
  - Do not use **heat it** if you have a reduced sensitivity to heat (e.g., due to polyneuropathy).

- 7. Side-effects**
- The heat may feel unpleasant for some users, although most people tolerate it well.
  - Excessive use may result in skin injury or persistent discoloration of the skin („coloric hyperpigmentation“).
  - Sensitive people may experience a temporary redness of the skin after use. In rare cases, users with sensitive skin may experience irritation.

- 8. Warnings**
- Using **heat it** is not a substitute for medical diagnosis or treatment. **heat it** is not suitable for treating allergic (anaphylactic) reactions. In case of any kind of pain or illness and if in any doubt, consult a doctor.
  - **heat it** cannot prevent disease transmission or cure symptoms or diseases that are not listed in the intended use. If you suspect an insect-borne disease, consult a doctor immediately.
  - If you have any doubts about the use of **heat it**, be sure to consult a doctor.
  - Do not treat the same skin area more than five times within one hour.
  - A break of at least 2 minutes is compulsory between repeated treatment on the same location.
  - Never press **heat it** onto the skin during the preheating phase, and never apply it for more than 10 seconds.
  - Keep out of reach of children – risk of choking due to small parts.
  - In case of unusual behaviour of **heat it**, do not use **heat it** again and contact the manufacturer. Please also report any serious incidents that occur in connection with **heat it** to the competent authority in your country.
  - Do not open or modify **heat it** and do not use if damaged.
  - Do not charge the smartphone under any circumstances while using **heat it**. Otherwise, contact with dangerous voltage cannot be ruled out.

Apple, iPhone and Lightning are trademarks of Apple Inc. registered in the U.S. and other countries. Use of the Made for Apple badge means that an accessory has been designed to connect specifically to the Apple product(s) identified in the badge and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with an Apple product may affect wireless performance.

Android and Google Play are trademarks of Google LLC. Design and specifications are subject to change without notice.

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